Wart-X

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Wart-X

Wart-X Wart & Verruca remover uses microdosing for the fast & effective removal of warts and verrucas. Only one treatment per week is needed. Can be used on children as of 4 years old.

Read the instruction for use prior to treatment. Use only as directed.

Background on Warts & Verrucas:

Warts are harmless growths on the skin, caused by the human papilloma virus (HPV). There are different types of warts but the most prevalent are called common warts. They consist of a core of inner tissue supplied by blood vessels, covered with layers of outer skin tissue.

Common warts are recognizable by the rough, "cauliflower-like" appearance of the surface. Verrucas are similar to common warts but mainly appear on the soles of the feet, heels or toes. They tend to be flatter and are often painful due to the pressure caused by walking.

Warts & verrucas affect 7 to 10% of the general population. They are very contagious and may appear more often if the body's defense system is weak. They can be passed from person to person or from one body location to another by direct contact with the infected area. More than 50% of warts disappear within two years with no treatment. However, many people choose to treat them at the onset as they can be unsightly and, in the case of verrucas, painful.

Traditionally there have been two main ways to treat warts and verrucas. Firstly, acid based liquid preparations that are applied directly onto the wart or verruca; and secondly, removal with laser therapy, burning or cryotherapy, either by a medical professional or by using a self treatment at home. The Wart-X Wart & Verruca remover is an effective and easy to use self treatment containing a highly concentrated liquid to remove warts & verrucas.

Warnings & Precautions:

TCA is a strong corrosive acid.

- Not suitable for children under 4.
- Not recommended for patients suffering from diabetes or poor circulation.
- Use only for the removal of common warts and plantar warts (verruca).
- For single patient use only.
- For external use on hands and feet only. Do not use on other body parts.
- Do not use on open wounds or irritated skin.
- If you are not sure it's a wart.
- Keep out of reach of children.

Discard the device in the following cases:

- The packaging or product is damaged.
- The tip of the device does not break when turning the cap upon first usage.
- No audible sound is produced upon clicking the blue button.

Please consult your doctor or dermatologist in case:

- The product will be used in combination with other treatments.
- You experience side-effects or allergic reactions occur.
- The symptoms don't improve after 6 weeks of treatment.
- Of pregnancy or breastfeeding.

Ensure disposal in line with international and national waste regulations.

Keep this instruction leaflet, this contains important information.


Do not open the cap before having rotated it! (see instruction for use, point 1)


Avoid contact of the TCA solution with healthy skin and eyes. In case of contact, wipe off the solution immediately with a dry cotton swab to minimize peeling of the healthy skin. If the product gets into the eyes, flush with water for 15 minutes and seek medical help immediately.

Instructions for Use


For instructions for use & additional info →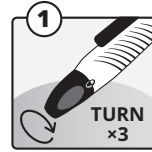


For first time usage and before removing the cap: rotate the cap 360° at least 3 times. This is required to break the inner seal. If this instruction is not followed, the pen won't be activated and will not be suitable for use.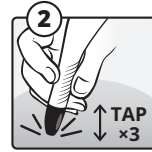


Tap with the pen (cap facing down) 3 times on a flat solid surface (e.g. a table) so the liquid can flow into the tip of the pen.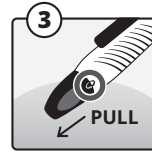

Align the blue bump (on the cap) with the white bump (on the pen) and pull the cap from the pen. The cap is childproof so some force might be needed.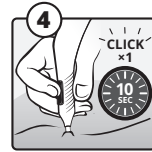

Place the tip of the pen on the wart. Now click the blue button once and keep the tip of the pen on the wart for **10 seconds**. A microdose (the size of a poppyseed) of the transparent wart removal liquid will be released from the tip of the pen. In case of larger warts (> 6mm) an additional application may be required.

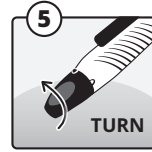After the application, replace the cap. Rotate the cap and make sure the blue and white bumps are not aligned (childproofing).

Repeat the application once per week until the wart has disappeared or for a maximum of 6 weeks. For the treatment of up to 3 warts. For each next application, repeat steps 2-5, including tapping the pen (cap facing down) 3 times on a flat solid surface before each use.

For better results, we recommend to soak the feet or hands in warm water for 5 minutes prior to application. After doing so, make sure to towel dry the wart before applying the product.

Side Effects:

- During the treatment period, the wart may become dark. This is normal and part of the healing process.
- A slight tingling sensation might be noticeable. This is normal and transient. If the discomfort persists in the following days, discontinue the treatment and consult a doctor.
- To avoid irritation and hyperpigmentation, treated skin should be protected from the sun until 30 days after treatment.
- Once opened, the product must be used within 6 weeks.

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TAV Medical Ltd.